

# LINEAR CUTTER STAPLER SINGLE USE

1

Remove the device from the packaging (it will be in two parts). Open the handle of the device fully, as it will be in the intermediate position, otherwise you will not be able to close the device when required.




2

Check the reload is correctly seated in the housing and has not been moved during transit.

3

Remove the staple protection tab ONLY when the device is about to be fired.

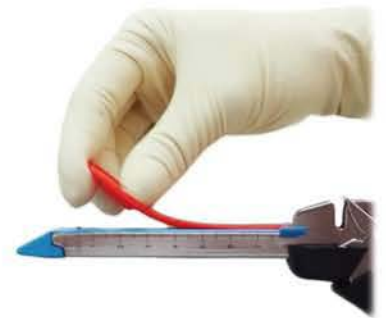

4

When placed over tissue, hold the device closed for 10-15 seconds, prior to firing.

5

Fire the device, initiating the firing knob all the way to the end of the track



6

Pull back the firing knob completely to ensure it has returned into a fully retracted position



## NOTES

- Cartridge sizes can accommodate the following tissue thickness; Blue = 1.5 mm, Green = 2.0mm
- The device can be reloaded a further 7 times, making a total of 8 firings
- The device is available in a 60mm and 80mm. All reloads are interchangeable within the same size group.