



Advanced Technology for Vets and Pets

INSTRUCTIONS FOR USE

Infiniti Medical® Drainage Catheter

CAUTION: U.S federal law restricts this device to sale by or on the order of a veterinarian.

CAUTION: This product is intended for veterinary use only. It is not for human use.

TARGET SPECIES: Canine

CAREFULLY READ ALL INSTRUCTIONS PRIOR TO USE: FAILURE TO FOLLOW SPECIFIC INSTRUCTIONS WARNINGS AND PRECAUTIONS MAY RESULT IN PROCEDURE COMPLICATIONS. AN ASEPTIC TECHNIQUE IS REQUIRED.

DEVICE DESCRIPTION

The *Infiniti Medical® Drainage Catheter* with hydrophilic coating is a radiopaque catheter with a multi side hole distal pigtail tip used for percutaneous drainage. The Drainage Catheter can be placed over or through a peel away sheath system. It contains an adjustable suture wing to facilitate securing the device after placement.

AVAILABLE SIZES

The *Infiniti Medical® Drainage Catheter* with hydrophilic coating is available in 2.5 Fr x 15 cm.

STORAGE

Store in a dry, dark, cool place.

INTENDED USE

The *Infiniti Medical® Drainage Catheter* is designed for percutaneous drainage of the kidney as a nephrostomy tube or for drainage of fluid from other body cavities. This product is intended for use by veterinarians appropriately trained and knowledgeable in percutaneous renal interventions as well as with the management of animals with kidney disease. It is for veterinary use only.

CONTRAINDICATIONS

Use of the introducer is contraindicated in patients with known or suspected coagulopathy and in patients with anatomy not suitable for the percutaneous placement of a catheter into the renal pelvis.

WARNINGS

- Do not use catheters for feeding tube/gastrostomy procedures. Exposure to gastric juices may damage the catheter.
- Do not allow alcohol or other solvents to come in contact with the catheter. Exposing the catheter to alcohol or solvents may damage the coating and the catheter.
- Pet owners should be advised that anesthesia carries a risk of serious and irreversible injury including death.

POTENTIAL ADVERSE EVENTS

Potential complications include risks normally associated with percutaneous interventional procedures. Other complications include, but are not limited to:

- Catheter occlusion
- Catheter dislodgement
- Hemorrhage
- Sepsis and urinoma/biloma
- Intestinal perforation
- Peritonitis
- Pneumothorax
- Skin infection
- Organ injury
- Vascular injury
- Death or serious injury related to stent placement or anesthesia required for the procedure.

PRECAUTIONS

- The sterile packaging and device should be inspected before use. If sterility or integrity of the device is suspected to be compromised, it should not be used.
- Do not autoclave or resterilize
- Do not expose the delivery system to organic solvents (e.g. alcohol).
- During insertion avoid contact with bone, cartilage and scar tissue, which may damage the catheter tip.
- This device is intended for single use only.
- Do not use continuously for more than 30 days.
- Do not expose to organic solvents, ionizing radiation or ultra violet light.
- Use prior to the sterilization expiration date on package label.
- Handle the product carefully and avoid forceful stretching etc.
- Use this product immediately after opening the package. After use, deposit it in a safe manner.

NOTICE

The material contained in this document does not describe the procedure for using this product and is not intended as a reference document for that procedure.

PRODUCT RECOMMENDATIONS

The *Infiniti Medical® Drainage Catheter* is intended to be used with the *Infiniti Medical® Renal Access Kit*.

INSTRUCTIONS FOR USE

Use sterile technique. Peel open package and place contents on sterile field. Prep skin and drape in area of anticipated needle puncture as desired.

Over the wire technique

1. Insert 21-gauge introducer needle into the renal pelvis along the least vascular plane with the assistance of ultrasound and or fluoroscopic guidance. Remove inner cannula. The needle position should be verified with imaging and by the ability to aspirate urine. Note that infected urine or urine contaminated with debris may be difficult or impossible to aspirate through the needle.
2. Remove the syringe and insert soft tip of the .018 in. outside diameter guide wire through the introducer needle into the renal pelvis. Advance guide wire to required depth so that the tip is securely in the renal pelvis. Leave an appropriate amount of guide wire exposed. At no time should the guide wire be advanced or withdrawn when resistance is met. Determine the cause of resistance before proceeding.
3. Hold the guide wire in place and remove introducer needle. Do not withdraw the guide wire back into the cannula as this may result in separation of the guide wire. **Caution:** If the guide wire must be withdrawn while the needle is inserted, remove both the needle and wire as a unit to prevent the needle from damaging or shearing the guide wire.
4. Dilate the soft tissue tract and advance the *Infiniti Medical® Drainage Catheter* over the wire and position pigtail loop in the renal pelvis.
5. While holding the catheter in place, remove guide wire and secure catheter in place.

Peel away technique

1. Peel open package and place contents on sterile field.
2. Prep skin and drape in area of anticipated needle puncture as desired.
3. Insert 21-gauge introducer needle into the renal pelvis along the least vascular plane with the assistance of ultrasound and or fluoroscopic guidance. Remove inner cannula. The needle position should be verified with imaging and by the ability to aspirate urine. Note that infected urine or urine contaminated with debris may be difficult or impossible to aspirate through the needle.
4. Remove the syringe and insert soft tip of the .018 in. outside diameter guide wire through the introducer needle into the renal pelvis. Advance guide wire to required depth so that the tip is securely in the renal pelvis. Leave an appropriate amount of guide wire exposed. At no time should the guide wire be advanced or withdrawn when resistance is met. Determine the cause of resistance before proceeding.
5. Hold the guide wire in place and remove introducer needle. Do not withdraw the guide wire back into the cannula as this may result in separation of the guide wire.
6. **Caution:** If the guide wire must be withdrawn while the needle is inserted, remove both the needle and wire as a unit to prevent the needle from damaging or shearing the guide wire.
7. Thread the Peel Away assembly over the guide wire.
8. Advance the Peel Away assembly over the guide wire and into the renal pelvis. Fluoroscopic observation may be advisable.
9. Remove the inner dilator leaving the outer catheter of the Peel Away Sheath and guide wire in place.
10. Thread the drainage catheter over the guide wire and advance the catheter through the Peel Away Sheath until the leading loop is in the renal pelvis.
11. Slowly withdraw and remove the guide wire allowing the pigtail to form in the renal pelvis.
12. Remove the Peel Away sheath by splitting the catheter and removing carefully.
13. Secure the drain in place.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. The product is intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REUSE PRECAUTION STATEMENT

For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure, which in turn, may result in patient injury, illness, or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

LIMITED WARRANTY

Infiniti Medical warrants to Buyer that products supplied by Infiniti Medical that are sold to Buyer will be free from defects in material and workmanship for six (6) months after delivery to Buyer. Buyer must inspect and notify Infiniti Medical of any such defects within this six (6) month period. Further, notice of a defective product must be given to Infiniti Medical in writing within ten (10) days following the discovery of such defect prior to the expiration of the warranty period in order to recover under the warranty. All returns are subject to the prior authorization of INFINITI, in its discretion.

The warranty does not cover and Infiniti Medical will have no warranty obligation whatsoever with respect to any damage to a product caused by or associated with: (i) usage not in accordance with product instructions or usage for a purpose not indicated on the labeling; (ii) abuse, misuse, neglect, improper maintenance or storage, accident, vandalism, or the negligence of any party other than Infiniti Medical; (iii) external causes, including (but not limited to) natural disasters, acts of God, power failure, cosmetic damage or damage to product packaging; or (iv) use of unauthorized consumables and/or accessories with the product. Infiniti Medical's sole liability under this warranty will be, at Infiniti Medical's sole option, to a) replace; b) repair; or c) refund the purchase price of the defective product(s). This will be Buyer's exclusive remedy for a covered defect. Any oral or written statement concerning the products inconsistent with the limited warranty set forth herein will be of no force or effect.

INFINITI EXPRESSLY DISCLAIMS ALL OTHER WARRANTIES AND CONDITIONS, EXPRESS OR IMPLIED, STATUTORY OR OTHERWISE, INCLUDING, WITHOUT LIMITATION, WARRANTIES OR CONDITIONS OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AS WELL AS ANY WARRANTIES ARISING FROM COURSE OF DEALING AND USAGE OF TRADE, AND INFINITI MEDICAL DOES NOT REPRESENT OR WARRANT THAT ANY PRODUCT WILL MEET BUYER'S REQUIREMENTS.

RETURNS

Buyer must notify Infiniti Medical within seven (7) calendar days of delivery regarding any products delivered to Buyer that were shipped in error, were damaged in shipping, or were in a shipping package that was damaged in shipping and such damage to the shipping package may have affected the quality of the products inside the shipping package.

Any products which Buyer wishes to return due to a) being shipped in error or damaged in shipping or b) a defect subject to the warranty provisions will be subject to receiving a Return Material Authorization (RMA) from Infiniti Medical. All returns are subject to the prior authorization of Infiniti Medical in its discretion. Only items appearing on an approved RMA are acceptable for return. Product returns will only be accepted from the original Buyer. Product returns will not be accepted from any third parties. Unauthorized returns will be destroyed and no credit issued. All authorized returned products must be shipped freight prepaid to the Infiniti Medical location indicated on the RMA, except Infiniti Medical will pay freight costs for product shipped-in-error or damaged in shipping.

LIMITATIONS OF LIABILITY

In no event shall Infiniti Medical be liable to Buyer for any unforeseen, indirect, incidental, special, punitive or consequential damages (including any loss of use, loss of revenue or damage for lost or anticipated profits), or otherwise arising out of or in connection with furnishing of products or service hereunder, or the performance, use of, or inability to use any products or service, or otherwise, whether based in contract, warranty, tort, including without limitation, negligence and strict liability, or any other legal or equitable theory. Infiniti Medical's total liability for any claim or action, whether based in contract, warranty, tort, including without limitation, negligence and strict liability, or any other legal or equitable theory shall not exceed the purchase price of the product or products out of which such claim or action arose, or Ten Thousand Dollars (\$10,000.00), whichever is less.

All material copyright © 2019 Infiniti Medical. All rights are reserved. No part of any material on this page may be reproduced, or stored in a database or retrieval system, distributed, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise, without the prior written permission of Infiniti Medical.